


Correct Loading of Applicators


QuickTag
make it your trademark

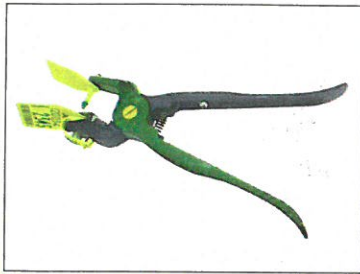

1. Please ensure you have a bespoke TypiFix™ Applicator as pictured.
2. Separate the male and female tag parts.
3. Slide the male tag part onto the pin of the applicators as far as it will go.
4. Press the sample container on the female part of the tag onto the socket on the front of the female flag.



5. Insert the folded sample container/female part of the tag deep into the applicator cavity under the retainer clip (sample container faces outwards).
6. Fold the strap of the sample container back and affix onto the brass bolt on the outside of the applicator jaw.
7. Gently close the applicators and make sure the conical punch on the male tag aligns within the centre of the marked circle on the female tag part.
8. The tag is now ready to be applied.
9. With the animals head restrained locate a central point on the ear free from major veins. PLEASE ensure that the part of the ear being penetrated is dry and free from dirt.
10. Position the applicators with the female part of the tag inside the ear and squeeze firmly together to ensure satisfactory locking and correct fit.
11. If the sample container was affixed correctly it will detach automatically from the tag when drawing back the applicators.

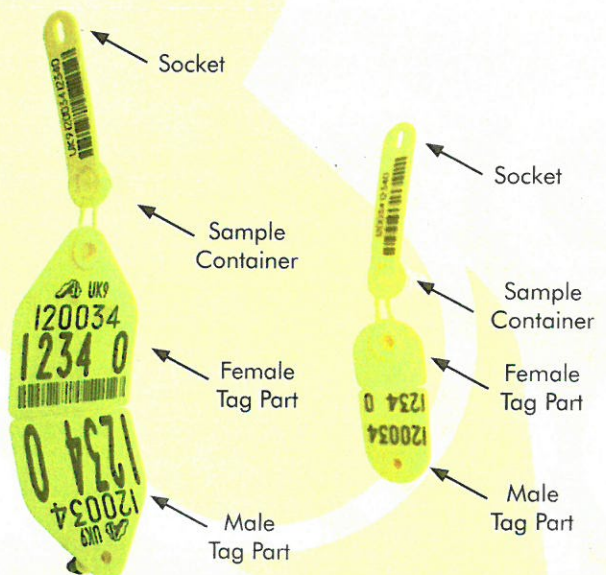

www.quicktag.co.uk